


Brava[®] Protective Seal

Sizing Guide

With 12 distinct sizing options, our **Brava[®] Protective Seal portfolio has a size to fit every body. Which one is right for you?** Answering a few simple questions can help you decide!


1. What is the size of your stoma?
(inner diameter of seal i.e. stoma size)
2. How much coverage are you looking for?
(adhesive diameter)
3. What does the area around your stoma look like? *(adhesive diameter and thickness)*
 - a. flat to mostly flat *(standard, thin seal)*
 - b. some creases, bumps, and folds *(standard or wide seal, thin or thick)*
 - c. deep creases, bumps, and folds *(wide or extra-wide seal, thin or thick)*

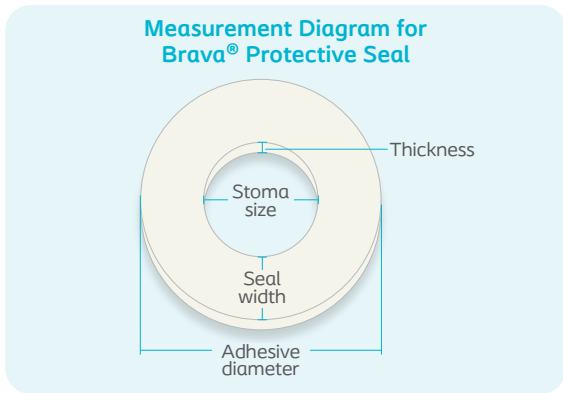

Standard Seals

For a **better fit** and **improved protection against leakage*** for skin with dips, creases or folds.

Stoma size | Adhesive diameter

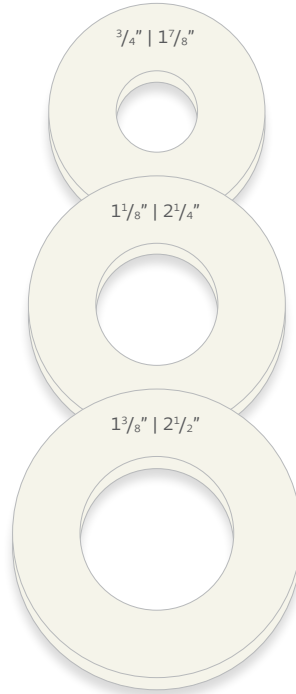

Available thicknesses:
2.5 mm (thin) 4.2 mm (thick)

Item number	Stoma size Adhesive diameter	Thickness
12035	3/4" 1 7/8"	thin
12045	3/4" 1 7/8"	thick
12037	1 1/8" 2 1/4"	thin
12047	1 1/8" 2 1/4"	thick
12039	1 3/8" 2 1/2"	thin
12049	1 3/8" 2 1/2"	thick

Wide Seals

Features greater adhesive area, for **increased coverage of peristomal skin** and a better fit over **skin with more challenges** such as deeper creases and folds.

Stoma size | Adhesive diameter



Available thicknesses:
2.5 mm (thin) 4.2 mm (thick)

Item number	Stoma size Adhesive diameter	Thickness
12036	3/4" 2 1/4"	thin
12046	3/4" 2 1/4"	thick
12038	3/4" 2 1/2"	thin
12048	3/4" 2 1/2"	thick

All come 10 units per box.

Prior to use, refer to the product 'Instructions for Use' for intended use and relevant safety information.

*Compared to not using a seal

Extra-wide Seals

Designed for **large uneven peristomal areas**. Provides the **most coverage** for increased feelings of security.

Stoma size | Adhesive diameter



Available thicknesses:
2.5 mm (thin)

Item number	Stoma size Adhesive diameter	Thickness
12032	3/4" 3"	thin
12033	1 1/8" 3"	thin

Have questions or want to learn more? Coloplast is here to help.

Reach out to us at **1-888-726-7872** or visit www.coloplast.us/bps